

dexcomG6™

START HERE

Set up



Your smart phone

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
Receiver

Transmitter

Sensor

G6 Overview

Your smart device




Dexcom
receiver




Display Device

- Shows glucose information
- Set up your smart device, Dexcom receiver, or both
- For a list of current compatible smart devices and operating systems go to: dexcom.com/compatibility

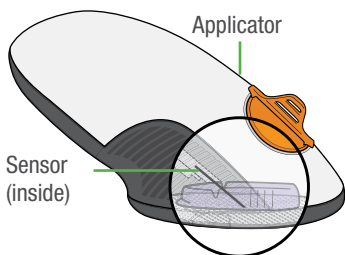

Applicator With Built-in Sensor

- Sensor gets glucose information
- Sensor applicator inserts sensor under your skin

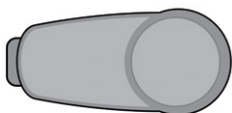

Transmitter


- Sends glucose information from sensor to display device

All graphics are representational. Your product may look different.

Review the Safety Statement in *Using Your G6*, Appendix E before using your G6.

What It Does

The G6 sends G6 sensor glucose readings (G6 readings) to your display device.

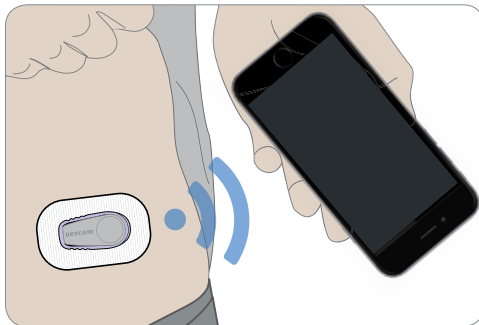

Use tabs below to set up the app or receiver.

Want to set up both? Pick one to set up first and turn to that tab. The last step shows you how to set up the second display device. Don't use both tabs.

For other ways to learn how to set up your G6:

- Watch the tutorial found in your product information bundle or online at: dexcom.com/links/g6/tutorial
- Contact Dexcom Care for individualized support or to register for free, online webinars at: dexcom.com/dexcom-care or 1.888.738.3646
- For Technical Support, email TechSupport@dexcom.com, or call 1.888.738.3646 (Toll free) or 1.858.200.0200 (Toll).



Your smart phone



Transmitter



Sensor

Set Up App

Step 1: Set Up App

A Download and open the Dexcom app




B Follow onscreen setup instructions

1 When prompted, enter your:

- Sensor code from the sensor applicator

No sensor code?

See *Using Your G6, Appendix A Troubleshooting*

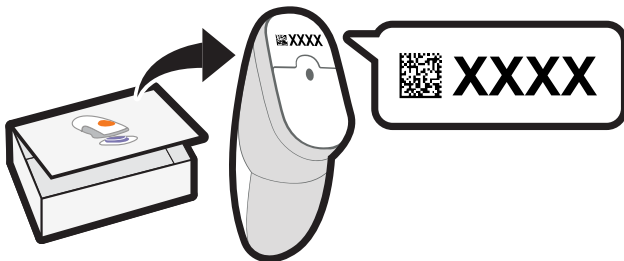

Serial Number (SN) from:

Transmitter box



or

Back of transmitter



- 2** See the blue sensor warmup timer?
That means your sensor is getting used to
your body.


During warmup:

- No G6 readings, alarm/alerts
- Keep smart device within 20 feet
of transmitter



C Wait 2 hours

- When complete, tap **OK** to see the home
screen
- Now you get G6 readings, alarm/alerts

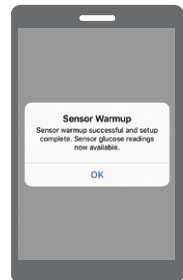

Step 2: See *Using Your G6*

Learn how to:

- Read your home screen
- Use alarm and alerts
- Make treatment decisions
- Troubleshoot issues




Step 3: Set up Receiver

Turn on the receiver by pressing and holding the power button down for 2-3 seconds. Then follow onscreen instructions.

Don't use the Set Up Receiver tab. Those steps are for setting up the receiver before setting up the app.



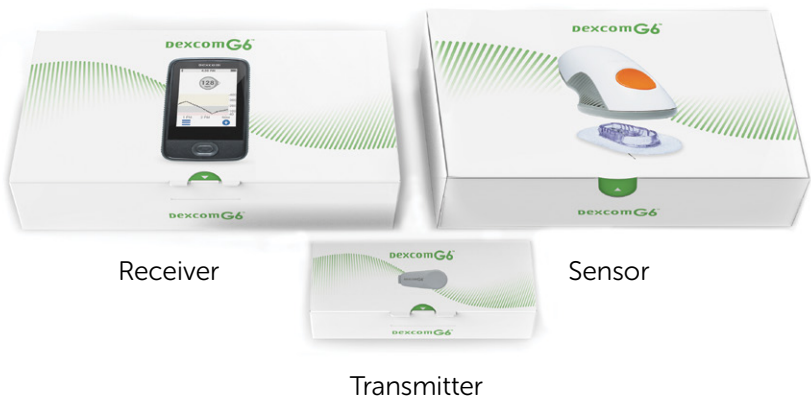

Set Up Receiver

Step 1: Set Up Receiver

A Take receiver out of box




B Turn on receiver

Press and hold power button for
2-3 seconds



C Follow onscreen instructions

- 1 When prompted, enter your:
 - Sensor code from the sensor applicator you will insertNo sensor code?
See Using Your G6, Appendix A Troubleshooting

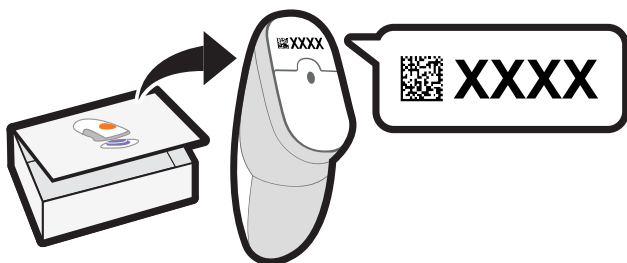

Serial Number (SN) from:

Transmitter box




or

Back of transmitter



2 You're done with this step when you see this screen:

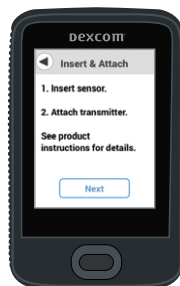


Step 2: Use Applicator to Insert Built-in Sensor

- A** Take applicator with built-in sensor out of sensor box




- B** Choose sensor site

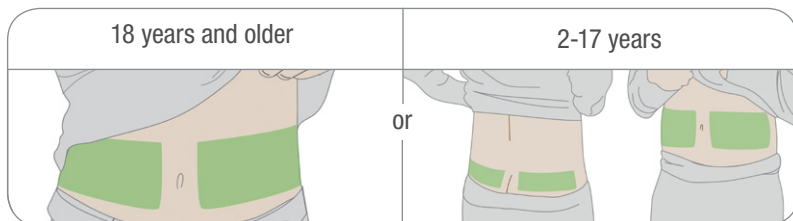


- C** Use applicator to insert built-in sensor


- 1** Wash and dry hands; clean sensor site with alcohol wipe

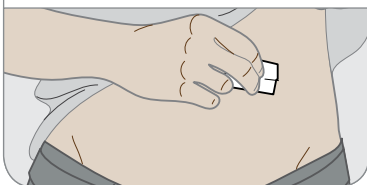


- 2** Use applicator with sensor code entered during display device setup

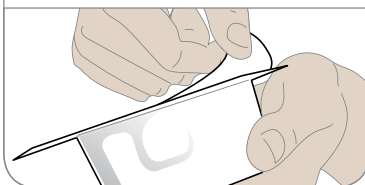


3 Remove both labels; don't touch adhesive

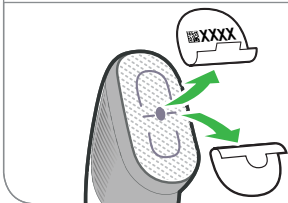


4 Move applicator to sensor site

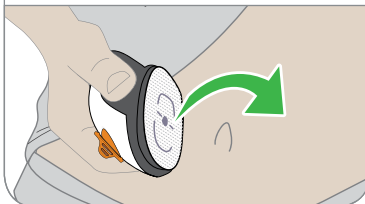

5 Place adhesive on skin

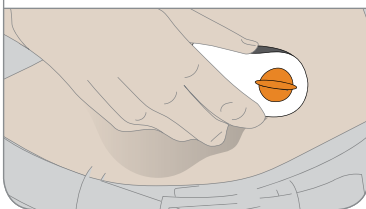


6 Fold and break off safety guard

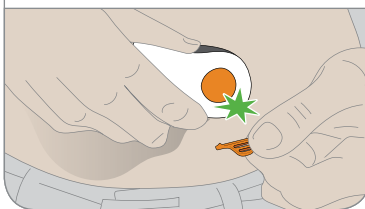

7 Press button to insert



8 Discard applicator

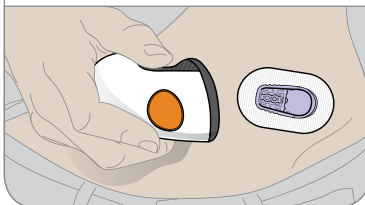


Step 3: Attach Transmitter

A Take transmitter out of box




B Snap in transmitter


1 Clean transmitter with alcohol

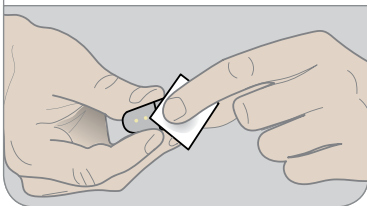


2 Insert transmitter tab into slot

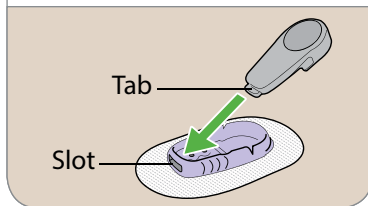

3 Snap transmitter firmly into place

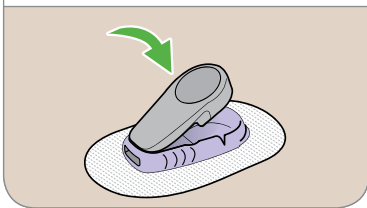

4 Rub around patch 3 times

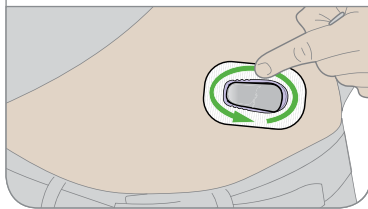

Step 4: Start Sensor on Receiver


A Wait up to 30 minutes for pairing



B Tap Start Sensor to begin 2-hour warmup

During warmup:

- No G6 readings, alarm/alerts
- Keep receiver within 20 feet of transmitter

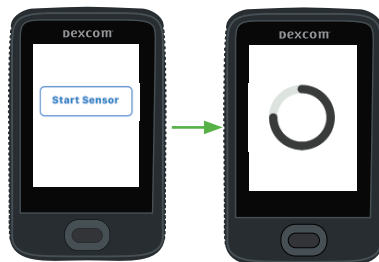

C Wait 2 hours


- When complete, tap Next to go to home screen
- Now you get G6 readings, alarm/alerts



Step 5: See *Using Your G6*

Learn how to:

- Read your home screen
- Use alarm and alerts
- Make treatment decisions
- Troubleshoot issues

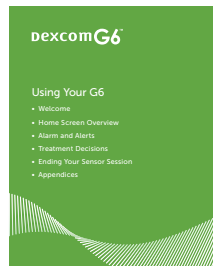

Step 6: Optional - Set up App

Download the app onto your smart device and open it. Then follow onscreen instructions.

Don't use the Set Up App tab. Those steps are for setting up the app before setting up the receiver.



Page intentionally left blank

Page intentionally left blank



© 2018 Dexcom, Inc. All rights reserved.

Covered by patents dexcom.com/patents.

Dexcom, Dexcom G4 PLATINUM, Dexcom G5 Mobile CGM System, G6, Dexcom Share, Dexcom Care are either registered trademarks or trademarks of Dexcom, Inc. in the United States and/or other countries. *Bluetooth* is a registered trademark owned by *Bluetooth* SIG. Apple is a registered trademark owned by Apple Inc. Android is a registered trademark owned by Alphabet Inc. All other product or company names that may be mentioned in this publication are trade names, trademarks, or registered trademarks of their respective owners.

Dexcom®



Dexcom, Inc.

6340 Sequence Drive

San Diego, CA 92121 USA

Phone: **1.858.200.0200**

Tech Support: **1.888.738.3646**

Web: dexcom.com